IM Portable Pro Unit (PPU)
Set Up Instructions

interactive metronome

1 THE PPU

Locate the PPU.


The top of the PPU has:

- · Mini USB Port
- Audio Input

The bottom of the PPU has:

- Wired Trigger Port
- Audio Port

The Indicator Lights are located in the center of the PPU.

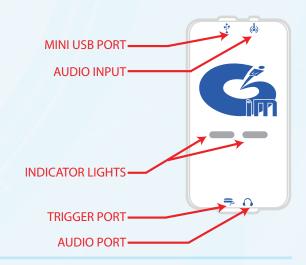

2 CONNECT THE TRIGGERS









The lights on the PPU will blink **RED** when the button trigger is pressed and blink **WHITE** when the tap mat is touched.

3 PLUG IN THE HEADPHONES

Plug the headphones into the audio port marked with a picture of headphones on the bottom of the PPU

DO NOT FORCE THE HEADPHONES IN!

IM Portable Pro Unit (PPU) Set Up Instructions



4 CONNECT THE USB & AUDIO CABLES TO THE PPU





5 CONNECT THE USB & AUDIO CABLES TO THE COMPUTER



The plug will only fit one way, do **NOT** force the USB cable in the wrong way!

HEALTHSOUTH

6 PPU READY TO USE



 If that does not work, make sure computer is NOT running off of battery power & unplug and

replug USB cable into computer.

available port.